


Put air into the lungs and you see a Colored Bar

Vertical Bars show Ventilation Rate and Ventilation Volume; Horizontal Bar Shows Interval Time

Target Color: **Bright Green** = everything is correct

Target Volume: Between the two green lines

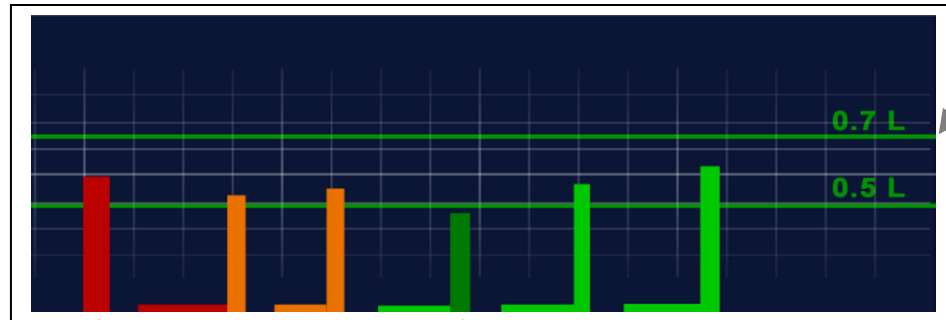

Vertical Red =

Ventilation rate too slow

Horizontal Red = Interval too long

Vertical Orange = Ventilation rate too fast

Horizontal Orange = Interval too short

Dark green =

Ventilation rate correct
but short on volume

Vertical Bright Green =

Correct ventilation rate & volume

Horizontal Bright Green =

Correct interval