

## SmartMan Code User Manual – 4.0 Running The Code

---

For SmartMan Code, Megacode and Megacode ROC  
V1.1.0 current at 1 October 2013

### Table of Contents

SmartMan Code User Manual – 4.0 Running The Code .....	1
SMARTMAN CODE SMARTMAN MEGACODE SMARTMAN MEGACODE ROC .....	1
4.0 HOW TO RUN THE CODE SOFTWARE .....	2
4.1 SELECT THE ACTIVITY.....	2
4.2 KNOW BEFORE YOUR RUN THE CODE.....	2
4.2.1 Length of Code.....	2
4.2.2 How to Start The Code .....	2
4.2.3 How to Stop The Code and Save Results .....	2
4.2.4 If Code Stops By Itself No Results are Saved.....	2
4.2.5 Indicate a Critical Event .....	2
4.2.6 Print and After Action Report (AAR) .....	3
4.3 THE CODE SCREEN.....	3
4.3.1 Information Area (During Code) .....	3
4.3.2 Real Time Feedback Area (During Code).....	3
4.3.3 Health Status Area (During Code) .....	4
4.3.4 Influencing the Health Score.....	4
4.3.5 Maintenance Level.....	4
4.3.6 Target Level for Health Status.....	4
4.3.7 Timer (During Code).....	4
4.3.8 Intervention Buttons (During Code) .....	4
4.3.9 Current Progress Area (During Code) .....	6

---

SMARTMAN CODE  
SMARTMAN MEGACODE  
SMARTMAN MEGACODE ROC

This is a training activity to help you improve how several people perform in their response to a Sudden Cardiac Arrest event. This normally is run with a group of between 3 and 6 people and one person is usually nominated as Team Leader, or Head Nurse, or Code Nurse, etc.

SmartMan Megacode is designed to give maximum flexibility to how the code is run. You coordinate the roles and SmartMan will provide an accurate time line of critical events. It also will provide an unsurpassed accurate detail related to how the CPR skills were performed.

SmartMan Code Training is also the only product in the world that is providing best practice training for AHA and ROC Protocols.

### 4.1 SELECT THE ACTIVITY

#### AHA or ROC

You have one or two options on this page depending on which version you have purchased.

NOTE: In order to have an accurate display for the ROC protocol you must have purchased a ROC protocol Airway SmartMan ALS manikin, otherwise physical changes must be made if you only have the standard Airway SmartMan manikin.

---

### 4.2 KNOW BEFORE YOUR RUN THE CODE

In order to run the code there are a few things you need to know.

#### 4.2.1 Length of Code

The code can run for a maximum of 20 minutes.

#### 4.2.2 How to Start The Code

Start the program when the code is called. You will see the timer is running (*middle*). The program must have something happen (such as a compression performed) within the first 40 seconds or it will terminate.

#### 4.2.3 How to Stop The Code and Save Results

Stop the code at any time by clicking on the redish STOP Button at the middle of the screen. You will only see the STOP Button when the code is running.

When you press the Stop Button, the program will stop. This will save the results and the program will create and display results. This means that you can now view what was done and you can create an After Action Report (AAR) by pressing the "Print" Button (*middle of page*).

#### 4.2.4 If Code Stops By Itself No Results are Saved

When the program will **stop** (time out)

- If you do not do anything within 40 seconds, it will stop. No results will be saved.
- If you do not press the Defib Button to Analyze within 10 minutes, it will stop. No results will be saved.

#### 4.2.5 Indicate a Critical Event

When a critical event such as establishing an IV or having the AED Analyze takes place you must press the corresponding button on the screen. You can click on the button or press the related keyboard key.

Perform your protocol, click on the button (*middle of the screen or press the key*)

[Top](#)


#### 4.2.6 Print and After Action Report (AAR)

When you have completed a code and the results are showing on the screen, press the “Print Results” Button near the middle of the screen.

---

### 4.3 THE CODE SCREEN

Once you start the Megacode, there are 5 separate areas which display information on what is being performed. You will see the following screen.

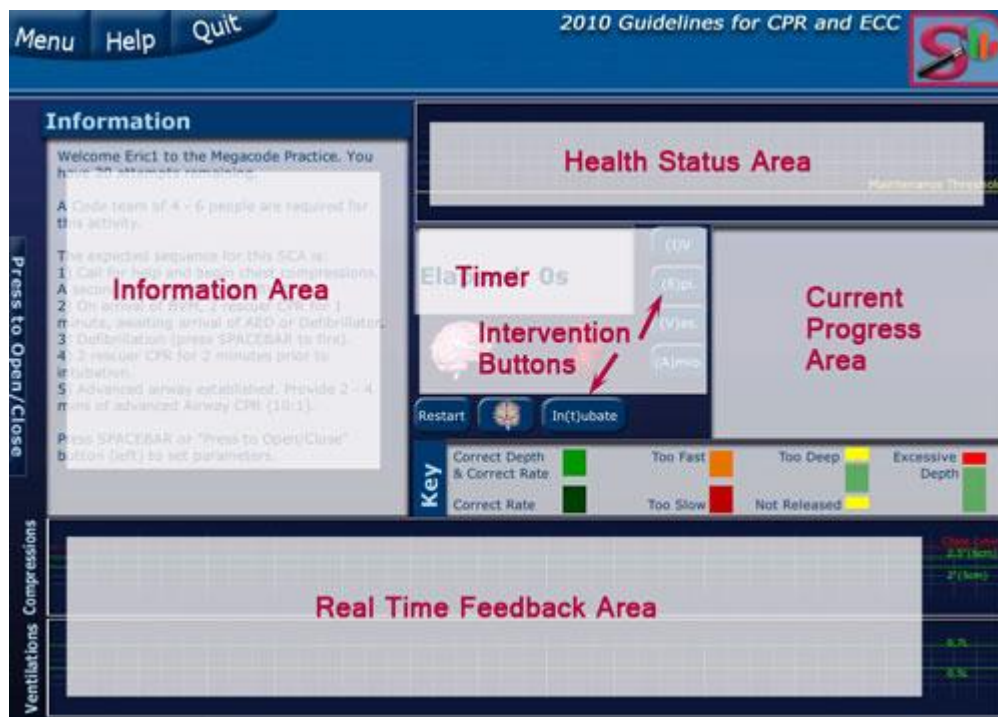

[Top](#)

#### 4.3.1 Information Area (During Code)

Before you begin, this area provides the description of what is expected in the activity. As you perform, this area will give you a status of which stages have been completed. After Stage 1, this information is displayed once you press the SPACE BAR to have the AED analyze.

#### 4.3.2 Real Time Feedback Area (During Code)

This is the bottom graph area and is active as you perform CPR. It provides an at-a-glance view of the quality of the chest compressions and the ventilations. The best performance for the patient is bright green. If every bar is not bright green then some aspect of the particular skills needs to be adjusted. The more bright greens produced, the greater the chance that the performance will have a positive influence on survival outcome.

[Top](#)

### 4.3.3 Health Status Area (During Code)

This area shows the relationship between chest compressions performed, the quality of those chest compressions and the time from onset. It provides an at-a-glance view of the current health of the patient. Think of this as representing “blood flow to the brain”.

### 4.3.4 Influencing the Health Score

You influence the health score by performing better compressions without interruptions. For either of these, the health score will drop. It will drop to 20% in 50 seconds.

If the Health Status is 20% or above you can maintain the status but not improve it.

If Health Status is below 20% you will see it rise with better compressions with no interruptions.

### 4.3.5 Maintenance Level

This shows as a line on the Health Status Area,

As time progresses the maximum value you can maintain via perfect chest compressions is indicated by the horizontal yellow line.

### 4.3.6 Target Level for Health Status

If the code has run longer than 5 minutes a health score of 20% is excellent. With high quality CPR you can keep the flow of blood of an SCA patient at 20% of a normal.

### 4.3.7 Timer (During Code)

This starts as soon as the code is called. It is cumulative in minutes and seconds and lets you know how long the code has been running. The various buttons will report when on the timeline the various interventions were performed (when you clicked on the corresponding button).

### 4.3.8 Intervention Buttons (During Code)

You can click on these buttons anytime during the code. The time these were administered will be shown in the results.

[Top](#)

Button	Description
Drugs Buttons	<p>Administer drugs at any time:</p> <ul style="list-style-type: none"><li>Epinephrine Click Button or press E Key</li><li>Vassopresin Click Button or press V Key</li><li>Amiodarone Click Button or press A Key</li></ul> <p>There is no maximum number of times you can press these buttons.</p> <p>In the on-screen report and in the printed report produced once the activity has ended, it will indicate which drug was given and when.</p>
IV Button	<p>Click on this Button or press the letter I on the keyboard to indicate that an IV line has been established.</p>

	<p>In the on-screen report and in the printed report produced once the activity has ended, it will indicate when the IV line was established.</p>
Intubate Button	<p>You can intubate the victim during any cycle after the first AED analysis.</p> <p>You do not have to intubate. You can continue CPR for every cycle without placing an advanced airway. Once you have an advanced airway, CPR changes to 10:1 ALS</p> <p>When the intubation is successful, click on the Intubate Button or press the letter t on the keyboard.</p> <p>In the on-screen report and in the printed report produced once the activity has ended, it will indicate when intubation has been done.</p>
STOP Button	<p>Stop the megacode activity any time by clicking the STOP Button.</p> <p>If you do not click on STOP you will not receive a report and you will not be able to print the report at a later time.</p> <p>Maximum time for the activity is 20 minutes. If the program reaches this time limit, the program will end and the results will be saved,</p>
Defib Space Bar	<p>Use of the AED is NOT a button. It is activated by pressing the SPACE BAR.</p> <p>It can be activated at anytime.</p> <p>When the SPACE BAR is pressed, a popup will indicate that the AED is analyzing. Although this popup will remain on the screen for 3 seconds, you can continue to perform chest compressions during if you wish.</p> <p>It is up to your trainer to determine whether it is a shockable rhythm (in which case you should wait for the AED to charge) or whether it is a non-shockable rhythm. You determine when you re-start chest compressions.</p>

[Top](#)

#### 4.3.9 Current Progress Area (During Code)

This area displays information on the number of compressions performed, number of CPR cycles, which cycle you are in, and which stage you are in.

As you move to the next cycle and the next stage information is updated.