

The Actions Menu

The Actions Menu is where you perform actions which are required by the program and view previous results of activities and tests performed by the person who is logged in.

Click on any area task in the Skills, Actions and Trainer menus to access the required action.

Actions Menu

1. Register: Every person who has logged in must be registered before the skills in the Actions and Trainer Menu are available.
2. Login Next User: This returns you to the login menu so that another person can log in to the program.
3. View Previous Results: This allows the person logged in to view previous results of activities and tests that they have completed. This menu also allows you to print results at a later time.
4. Check for Updates: you must be connected to the internet to use this feature. It will check to see if you have the latest version of the software or not.
5. Calibrate SmartMan: This is required before you use SmartMan the first time. You can calibrate at any time.
6. Activate SmartMan: Before you can use SmartMan, you must activate the program. To do this you must have a key code.
7. SmartMan ONLINE: If SmartMan is online, it will send results to the server. If it is not online it will not send results.

White or Grayed Out

Selections in the Actions Menu are only available when they are showing in dark blue font. For example, if the person logged in has already registered that option will not be available. It will be grayed out (lighter blue font).