

SmartMan Manual – 2. Actions Menu 6. Activate SmartMan

V5.1 and later

The Actions Menu

The Actions Menu is where you perform actions which are required by the program and you can view previous results of activities and tests performed by the person who is logged in.

Click on any item to perform the required action.

Actions Menu

1. Register
2. Login Next User
3. View Previous Results
4. Check for Updates
5. Calibrate SmartMan
6. Activate SmartMan
7. SmartMan ONLINE

7. SmartMan ONLINE

When using SmartMan, the user can select to send the results to a server online or to leave them on the local machine. To use the server you must have an active internet connection.

This button is user selectable.

ONLINE: In this mode the program will try and send results to the server as soon as a practice or a test results is completed. The results are also saved on your computer.

If this is set to ONLINE, and there is no internet connection, there might be a delay of several seconds when moving to display results as the program will try to establish a connection.

OFFLINE: In this mode the program will immediately display results when a practice or a test results is complete. The results are also saved on your computer.

If you decide to send results to the server at a later time, you can do this by selecting the 'View Previous Results Button' - select the particular practice or test result and click on the 'Continue' button in the 'Submit Results' pop up that appears. See more information under the 'View Previous Results' section.

Also, if you wish to submit a large number of results to the internet at one time, you can use a utility called the SmartMan Results Crawler. Contact SmartMan for more information.