

SmartMan Manual – 2. Actions Menu 2. Login Next User

V5.1 and later

The Actions Menu

The Actions Menu is where you perform non CPR skills actions which are required by the program.

Click on any item to perform the required action.

Actions Menu

1. Register
2. Login Next User
3. View Previous Results
4. Check for Updates
5. Calibrate SmartMan
6. Activate SmartMan
7. SmartMan ONLINE

2. Login Next User

What this does

This returns you to the Login page so that another person can log in to the program.

If the USB connection is lost, when you click on the message, you will be returned to the Login page.

Why you would use this

You would want to do this if another person is to perform skills. This means that their results will go under their own login. This provides them with a record of their scores anytime they come back and login.

You could do this in order to view another person's previous results providing you knew their login and password details.