


SmartMan Megacode



HEALTH BUTTON

SHOWING HEALTH STATUS

This is the bar which gives you an indication of how well your patient is responding to your CPR. It also records Critical Events.

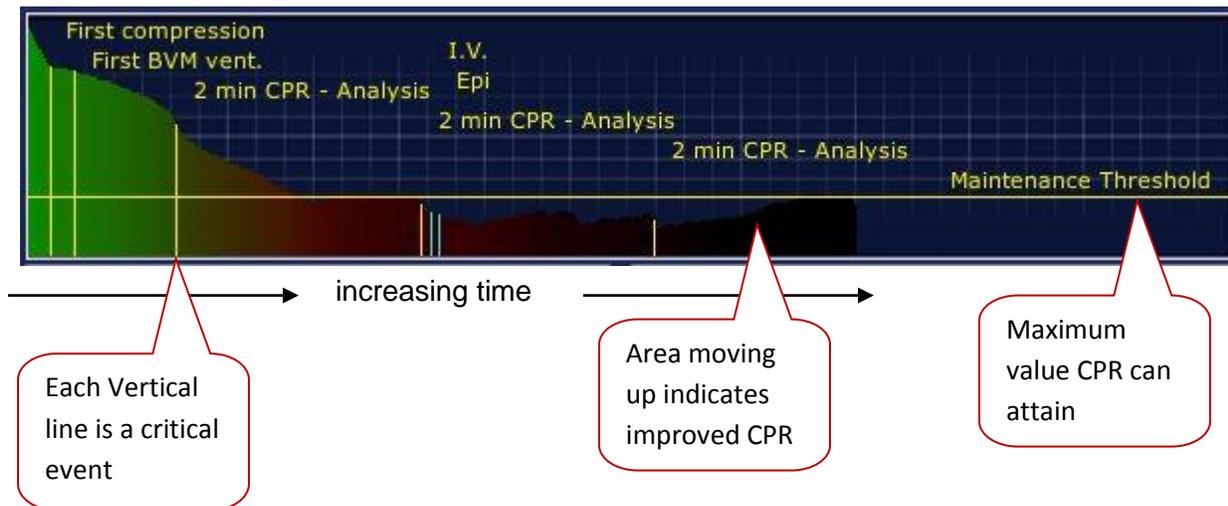

Think of Health Status as “blood flow to the brain”.
It will drop to 20% in 50 seconds.

How You Influence Health Status

You influence the health score by performing better compressions without interruptions.

- If the Health Status is above 20% you can maintain the status but not improve it.
- If Health Status is below 20% you will see it rise with better compressions with no interruptions.

Target Health Status

If the code has run longer than 5 minutes a health score of 20% is excellent.

MAINTENANCE THRESHOLD LINE

This is on the Health Status

With perfect CPR you can keep the flow of blood of an SCA patient at 20% of a normal patient.